ACCU-CHEK®

Safe-T-Pro Uno

Sterile, single-use lancing device

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REF 0588662, 0653971

Instructions for Use

Read these instructions for use before you obtain blood with the lancing device.

Intended Use

The lancing device is intended for obtaining capillary blood.

Indication

The lancing device is indicated for obtaining capillary blood for the determination of diagnostic parameters.

Intended User

Intended users are non-professional users 18 years and older and healthcare professionals.

Target Group

Healthcare professionals as user
Healthcare professionals can use the lancing device for adults, adolescents, children, and neonates.

Non-professional users
Adults can use the lancing device for themselves and other persons who are at least 3 years old. For children and adolescents (3 to 17 years), an adult caregiver must obtain the blood sample from the fingertip. Before the first use for children and adolescents, consult your healthcare professional.

Contraindications

Healthcare professionals as user
Blood must be obtained from the fingertip of children 1 year and older, adolescents, and adults.

Non-professional users
Blood must be obtained from the heel of children younger than 1 year and neonates.

Non-professional users
Blood must be obtained from the fingertip. Non-professional users must not obtain blood from children younger than 3 years.

Clinical Benefits

The penetration depth of 1.5 mm allows to obtain blood volumes for diagnostic purposes.

The specially beveled needle cut, the thin needle diameter, and the shortened needle avoid to obtain blood virtually pain free.

Warnings

There are no limitations

General Safety Information

The instructions for use contain warnings and precautions:
A WARNING indicates a foreseeable serious hazard.

A PRECAUTION describes a measure you should take to use the product safely and effectively or to prevent damage to the product.

WARNING

Risk of suffocation

This product contains small parts that can be swallowed.
Keep the small parts away from small children and people who might swallow small parts.

If the lancing device was dropped or exposed to other mechanical stress, check it for damages. If you suspect such as cracks, do not use the lancing device.

Device Description

See illustration 1

Release button

Warning

Sharply cut tip of the needle
The lancing device is a sterile product, which has been sterilized using irradiation.

Preparing to Obtain Blood

PRECAUTION

Risk of infection

A puncture site that was not cleaned can lead to infection.
Before obtaining blood clean the puncture site.

Non-professional users

Only for children 3 years and older, adolescents, and adults
Obtain capillary blood from the sides of the fingertips (see black areas in illustration 2). The sides of the fingertips are the least sensitive to pain compared to other areas of the finger. Any finger can be used for the puncture.

Wash the hands with warm water and soap and rinse thoroughly. This ensures that the puncture site is clean and it increases the blood flow. Dry the hands thoroughly.

Healthcare professionals as user

Children 1 year and older, adolescents, and adults
Obtain capillary blood from the sides of the fingertips (see black areas in illustration 2). The sides of the fingertips are the least sensitive to pain compared to other areas of the finger. Any finger can be used for the puncture.

Wash the patient's hands with warm water and soap, rinse thoroughly and let them dry completely. This ensures that the puncture site is clean and it increases the blood flow. If you are using alcohol wipes in addition, the finger must also be allowed to dry completely.

Children younger than 1 year and neonates

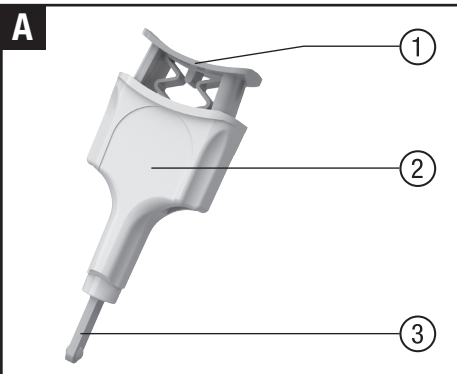
WARNING

Risk of injury

If the lancing device is used at the fingertip of children younger than 1 year and of neonates, it can injure the finger.

Only use the lancing device at the heel of children younger than 1 year and of neonates.

The capillary blood must be obtained from the heel. Obtaining blood must occur from the lateral areas of the sole (see black areas in illustration 3). The puncture must occur away from the sole (see areas in the enlargement). While obtaining blood, the child should be held with its foot vertically facing the floor. A child, which has already been punctured, must not be punctured a second time.

► To improve blood flow at the heel before obtaining blood, you can gently warm the heel, for example, by wrapping it in a warm, moist cloth at a maximum 42 °C.

► Wash the heel with warm water and soap, rinse it thoroughly and let it dry completely. Disinfect the heel with an alcohol wipe afterwards. Always let the heel dry completely.

Obtaining Blood

WARNING

Risk of infection

► If the sterility cap is damaged or missing, the needle can be unsterile and lead to infection.

► Usually inspect the lancing device. If the sterility cap is deformed, cracked, or otherwise damaged, do not use the lancing device. If the sterility cap is missing or any part of the needle is exposed, do not use the lancing device. Discard the lancing device.

► A used lancing device can transmit infections.

► Use a lancing device only once to obtain blood.

PRECAUTION

Risk of infection

► If the use by date of the lancing device has expired, the needle can be unsterile and lead to infection.

► If the use by date has expired, do not use the lancing device. The use by date is printed next to the 2 symbol on the packaging.

► Twist the sterility cap one quarter turn and pull it off.

► Hold the lancing device between the index finger, middle finger, and thumb.

► Press the lancing device firmly against the chosen puncture site.

► Using your thumb, press the release button down completely.

► Remove the lancing device from the puncture site.

► Wait for a blood drop to emerge from the puncture site.

► Finger: Gently massage the finger towards the fingertip.

► Heel: Exert slight pressure on the heel to encourage a sufficiently large blood drop to form.

For some diagnostic tests, the fingertip or heel must not be vigorously massaged or pressed, as doing so could produce incorrect test results.

WARNING

Risk of infection

Check that the needle does not protrude after you obtain blood. If the needle protrudes, do not touch the front of the lancing device. Discard the lancing device immediately.

Operating Conditions

Use the lancing device under the following operating conditions:
Temperature range: +5 to +45 °C
Relative humidity range: 5 to 90 %

Storing the Lancing Device

Keep the small parts away from small children and people who might swallow small parts.

If the lancing device was dropped or exposed to other mechanical stress, check it for damages. If you suspect such as cracks, do not use the lancing device.

Discarding the Lancing Device

WARNING

Risk of infection

If the lancing device has come into contact with blood, it can transmit infection.

Discard the used lancing device as infectious material in such a way that it does not cause injury to anyone. For example, place the lancing device into a sharps container.

► If you are a non-professional user, follow the regulations applicable in your country. For information on how to discard the lancing device correctly, contact your local council or authority.

► If you are a healthcare professional, follow your facility's disposal regulations.

Reporting of Serious Incidents

For a patient's third party in the European Union and in countries with identical regulatory regime, it, during the use of this device or as a result of its use, a serious incident has occurred, please report it to the manufacturer and to your national authority.

Last Update

2023-02

Customer Support

United Kingdom

Roché Diabetes Care Limited
Charles Avenue, Burgess Hill
West Sussex, RH15 5RY United Kingdom
Roché UK Customer Careline 11

UK Freephone number: 0800 701 700
ROI Freephone number: 1 800 709 600
www.accu-chek.co.uk

Malta

Waters Corporation
29, Santos Building
Viner Street
Marsa MSL1264
Malta MF20312

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