Instructions for Use

Intended Use

The Accu-Chek Insight Adapter & Tubing (transfer set) is part of an Accu-Chek Insight infusion set. The Accu-Chek Insight infusion set is intended for subcutaneous infusion of insulin with an Accu-Chek Insight insulin pump.

Device Description

- A Adapter
B Needle

Before You Get Started

Read these instructions for use, the instruction for use of the Accu-Chek Insight infusion set and the instructions for use of the Accu-Chek Insight insulin pump.

When to Use a New Transfer Set

Every time you insert a new insulin cartridge into your pump you need to replace the transfer set.


WARNING

Always use a new adapter when changing the cartridge.

The adapter is correctly positioned when it is aligned with the pump's housing. Insulin delivery is only guaranteed, if the needle is fitted onto the cartridge centrally in a perpendicular manner (see step 1).

Placing the Adapter on the Insulin Pump

- 1 After you inserted the cartridge, place a new adapter (A) on the bayonet socket of the cartridge compartment. Make sure you insert the nose of the adapter into the notches of the pump's housing at a right angle (90°). Otherwise the needle could bend.

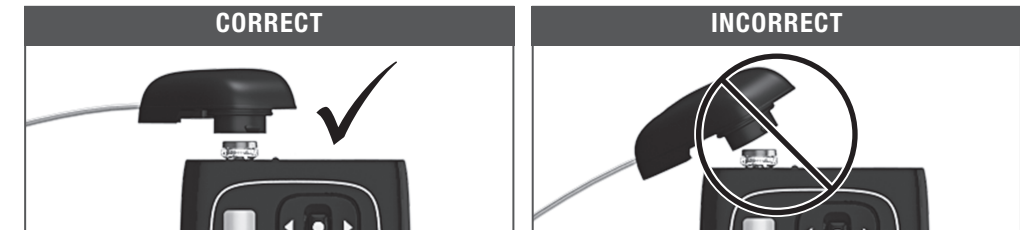- 2 Turn the adapter clockwise all the way until it stops. The adapter is correctly positioned when it is aligned with the pump's housing. Then press **OK** to confirm.
- 3 The Cartridge type screen appears. Press **OK** to confirm.
- 4 The pump detects the position of the plunger in the cartridge. The pump then performs a self-test.
- 5 After the self-test, the Fill tubing? screen appears. Press **OK** to fill the infusion set tubing with insulin.

For the next steps read the instructions for use of the Accu-Chek Insight insulin pump.

WARNING

If an empty cartridge is inserted into the pump (for example for training purposes), the pump displays the programmed insulin deliveries (infusion set priming, basal rate and bolus deliveries), even though no insulin is being delivered due to the empty cartridge.

Distributed in the United Kingdom by:

Roche Diabetes Care Limited
Charles Avenue, Burgess Hill
West Sussex, RH15 9RX, United Kingdom
Accu-Chek Pump Careline [®]
UK Freephone number: 0800 731 22 91
ROI Freephone number: 1 800 88 23 51

^{*)} calls may be recorded for training purposes
Some mobile operators may charge for calls to these numbers.

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