EN

Accu-Chek Insight Tender infusion set

The Accu-Chek Insight Tender infusion set is intended for the subcutaneous infusion of insulin with an Accu-Chek Insight insulin pump. The Accu-Chek Insight Tender infusion set consists of the head set (A-H) and the transfer set (I).

Package 1 contents (with transfer set):

- Head set (A-H)
- Transfer set (I)

Package 2 contents (without transfer set):

- Head set (A-H)
- Transfer set (I)

	Canula	Length: 13 mm	17 mm
A Needle cover	Fill amount (approx.): 0.9 U*	0.9 U*	0.9 U*
B Handle	9 µl	9 µl	9 µl
C Introducer needle			
D Connector plate			
E Adhesive plaster			
F Rear adhesive liner			
G Front adhesive liner			
H Disconnect cover			
I Transfer set (tubing)			

* 100U insulin

- Before unpacking your infusion set, wash your hands thoroughly. Choose and disinfect your infusion site as instructed by your doctor or healthcare team. Put back the needle cover until you hear an audible **CLICK**. This indicates that the needle cover is in place.
- Insert the infusion set at a 20° to 45° angle. Choose an infusion site away from your waistline, bones, any scar tissue, belly button and recent infusion sites. Rotate your infusion site regularly.
- Carefully peel the front adhesive liner away from the adhesive plaster.
- Press the front part of the adhesive plaster firmly onto the infusion site, ensuring that it makes good contact with the skin.
- While holding the adhesive plaster in place, pull the handle with the introducer needle sideways out of the connector plate.
- Push the needle cover back until it snaps into its original position and dispose of it.
- Peel the rear adhesive liner away from the adhesive plaster.
- Press the adhesive plaster firmly onto the infusion site, ensuring that it makes good contact with the skin.
- Attach the transfer set to the insulin pump and fill it according to the instructions provided with the insulin pump. Make sure it is completely filled and free from air bubbles.
- Connect the flexible tube to the connector plate. Before resuming insulin pump therapy, fill the head set and the soft canula with 0.9 U of insulin.

To disconnect the transfer set, separate the side wings of the connector together and withdraw it. Keep the connector plate clean. Attach the disconnect cover when appropriate.

Warning

- When using the infusion set for the first time, your doctor or healthcare team must be present.
- Check your blood glucose level at least 4 times a day, and at least once, 1 to 3 hours after inserting a new infusion set.
- If your blood glucose level should unexpectedly increase, or if an occlusion occurs, check for clogging or leaks. Change your infusion set immediately if you are not entirely sure that it is working properly.
- If the infusion site becomes inflamed, replace your infusion set at a new site.
- Change the head set at least every 3 days and the transfer set at least every 6 days, or as instructed by your doctor or healthcare team.
- Regularly check that your infusion set is properly attached to the skin to prevent accidental underdelivery of insulin.

Never fill the transfer set or attempt to free clogged tubing while the infusion set is connected to your body as you may infuse an uncontrollable quantity of insulin.

- Do not use this product for any medication other than insulin.
- Do not use this product if the blister package is already opened or damaged.
- Protect the product from excessive humidity, sunlight and heat. Store at room temperature.
- Discard your infusion set after use according to local regulations. To avoid infection and underdelivery of insulin, do not re-use it.
- Dispose of your infusion set/introducer needle safely, so that no one can be pricked or hurt by it, as this could lead to infection.
- Pay attention to the tubing of the infusion set connected to your body. There is a risk of strangulation if the tubing becomes wrapped around your neck. In case of doubt, use the shortest tubing length to minimise the risk of strangulation.
- This product contains small parts. There is a risk of suffocation if small parts (e.g. covers, caps or similar objects) are swallowed.

United Kingdom
Distributed in the United Kingdom by:
Roche Diabetes Care Limited
Charles Avenue, Burgess Hill
West Sussex, BN15 9RY, United Kingdom
Accu-Chek Pump Carline 15
UK Freephone number: 0800 731 22 51
RO Freephone number: 1 800 88 22 51
*1 calls may be recorded for training purposes
burgesshill@accu-chek.co.uk
www.accu-chek.ie

In US, distributed by:
Roche Diabetes Care, Inc.
Indevco, Inc. 46256, USA
Tel/Fax: 1-800-280-7801
ACCUCHEK Customer Care Service Center: 1-800-688-4578
www.accu-chek.com

Canada
Accu-Chek Pump Support: 1-800-688-4578 (toll free)
www.accu-chek.ca

Australia
Accu-Chek Enquiry Line: 1800 251 816
Pump Support: 1800 633 457
www.accu-chek.com.au

New Zealand
Accu-Chek Enquiry Line: 0800 80 22 99
Pump Hotline: 0800 696 696
www.accu-chek.co.uk

South Africa
Accu-Chek Customer Care Centre:
080-0486255 (Toll 080-34-22-38-37)
www.diabetes.co.za

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Roche Diabetes Care France SAS
PROPHAC S.A. r.l.