EN

Accu-Chek Insight Flex infusion set

The Accu-Chek Insight Flex infusion set is intended for the subcutaneous infusion of insulin with an Accu-Chek Insight insulin pump. The Accu-Chek Insight Flex infusion set consists of the head set (A-G) and the transfer set (H).

Package 1 contents (with transfer set):

• Head set (A-G)

• Transfer set (H)

Package 2 contents (without transfer set):

• Head set (A-G)

• Canula (H)

• Schlauch (H)

A Adhesive plaster	Canula			
B Adhesive liner	Lunghezza:	6 mm	8 mm	10 mm
C Connector plate	Capacità:	0,7 U*	0,7 U*	0,7 U*
D Introducer needle	(aprox.):	7 µl	7 µl	7 µl
E Needle box				
F Button	Tubo	40 cm	70 cm	100 cm
G Disconnect cover	Lunghezza:	40 cm	70 cm	100 cm
H Transfer set (tubing)	Capacità:	9 µl	13 µl	17 µl
	(aprox.):	90 µl	130 µl	170 µl

* U100 insulin

1

Before using your infusion set, wash your hands thoroughly. Choose and disinfect your infusion site as instructed by your doctor or healthcare team.

2

Remove the white liner on top (A), which will expose the blue liner. Then remove the blue liner (B).

3

Push the button until you feel the introducer needle with the soft canula snags out.

4

Insert the infusion set on top (A) angle. Choose an infusion site away from your waistline, bones, any scar tissue, belly button and recent infusion sites. Rotate your infusion site regularly.

5

Press the adhesive plaster firmly onto the infusion site, ensuring that it makes good contact with the skin.

6

Remove the needle box with the introducer needle at a right angle from the body, leaving the adhesive plaster and the connector plate in place on your skin. The introducer needle snaps back into the needle box automatically.

7

Attach the transfer set to the insulin pump and fill it according to the instructions provided with the insulin pump. Make sure it is completely filled and free from air bubbles.

8

Connect the filled transfer set to the connector plate. Before resuming insulin pump therapy, fill the head set and the soft canula with 0.7 U of insulin.

• Never fill the transfer set or attempt to free a clogged tubing while the infusion set is connected to your body as you may infuse an uncontrollable quantity of insulin.

• Pay attention to the tubing of the infusion set connected to your body. There is a risk of strangulation if the tubing becomes wrapped around your neck. In case of doubt, use the shortest tubing length to minimise the risk of strangulation.

• This product contains small parts. There is a risk of suffocation if small parts (e.g. covers, caps or similar objects) are swallowed.

United Kingdom

Distributed in the United Kingdom by:
Roche Diabetes Care Limited
Charles Avenue, Burgess Hill
West Sussex, RH15 9RY, United Kingdom
Accu-Chek Pump Careline (t)
UK Freephone number: 0800 731 22 91
ROI Freephone number: 1 800 88 23 51
ROI Freephone number: 1 800 88 23 51
1 calls may be recorded for training purposes
burgess@insulinpumps@roche.com
www.accu-chek.co.uk

In US, distributed by:
Roche Diabetes Care, Inc.
Indianapolis, IN 46256, USA
Toll Free: 1-800-280-7801
ACCUCHEK Customer Care Service Centre: 1-800-688-4578
www.accu-chek.com

Canada

Accu-Chek Pump Support: 1-800-688-4578 (toll free)
www.accu-chek.ca

Australia

Accu-Chek Enquiry Line: 1800 251 816
Pump Support: 1800 633 457
www.accu-chek.com.au

New Zealand

Accu-Chek Enquiry Line: 0800 80 22 99
Pump Hotline: 0800 696 696
www.accu-chek.co.nz

South Africa

Accu-Chek Customer Care Centre:
080-DIABETES (Dial 080-34-22-38-37)
www.diabetes.co.za

Warning

• When using the infusion set for the first time, your doctor or healthcare team must be present.

• Store at room temperature.

• Check your blood glucose level at least 4 times a day, and at least once, 1 to 3 hours after inserting a new infusion set.

• Do not use the product for any medication other than insulin.

• Do not use the product if the blister package is already opened or damaged.

• If the infusion site becomes inflamed, replace your infusion set at a new site.

• Change the head set at least every 3 days and the transfer set at least every 6 days, or as instructed by your doctor or healthcare team. Also consult the instructions for use of the insulin pump.

• Discard your infusion set after use, according to local regulations. Do not re-use it to avoid infection and underdelivery of insulin.

• Dispose of your infusion set and needle box safely, so that no one can be pricked or hurt by it, as this could lead to infection.

• Regularly check that your infusion set is properly attached to the skin to prevent accidental underdelivery of insulin.

• If your blood glucose level should unexpectedly increase, or if an occlusion occurs, check for clogging or leaks. Change your infusion set immediately if you are not entirely sure that it is working properly.

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