Gluing area

ACCU-CHEK® Solo

Reservoir assembly

Instructions for use

Front side

Roche

ACCU-CHEK®

C E 0123

Approved/listed/registered under the product name: Accu-Chek Solo micropump system

ACCU-CHEK and ACCU-CHEK Solo are trademarks of Roche.

© 2020 Roche Diabetes Care GmbH

Sandofer Strasse 116

68305 Mannheim, Germany

www.accu-chek.com

Last update: 2020-05

back side

Roche-LOT

Mat.-No. 0789350001

Gluing area

ACCU-CHEK® Instructions for use

ACCU-CHEK Solo Reservoir Assembly

Intended Use

The Accu-Chek Solo reservoir assembly (including the filling aid and the handle for the piston rod) is a part of the micropump in addition to the Accu-Chek Solo pump base. The reservoir is a sterile container for holding the insulin that the micropump delivers to the body.

Package Contents and Storage Conditions

See outer packaging.

Component Overview

A Reservoir
B Filling aid (blue)
C Handle for the piston rod (blue)
D Protective film for battery (orange)
E Reservoir needle
F Piston rod

Do not touch the needle of the filling aid (B) or the reservoir needle (E).

This product contains pointed and sharp-edged parts that may cause injury. Keep pointed and sharp-edged parts away from small children and vulnerable persons.

WARNING

Risk of suffocation

This product contains small parts that may be swallowed. Keep small parts away from small children and vulnerable persons.

Before You Get Started

Read these instructions for use and the instructions in the User's Manual of the Accu-Chek Solo micropump system. Take particular note of all warnings, information and notes required for the safe and proper use of the micropump system.

This is a sterile, single use product that has been sterilised with ethylene oxide. You can use the reservoir assembly for up to 4 days.

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several times.

8 Slowly push the handle upwards in the direction of the arrow to fill the reservoir with insulin.

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10 Align the piston rod (F) of the filled reservoir such that you can insert it into the piston rod opening of the pump base. Push the filled reservoir onto the pump base until both parts are tightly connected.

11 When both components are connected to each other correctly and the battery is activated, the micropump issues the Start-Tonfolge.

If this sequence of signals is not issued, check whether the battery is activated and repeat step 10.

12 Make sure that there is no gap between the reservoir and the pump base.

The pump base and the reservoir are only connected correctly when the Start-Tonfolge sequence is issued and when the pump base and the reservoir are seamlessly connected.

Disposal

WARNING

Risk of infection

Used components carry a risk of infection.

Dispose of the reservoir assembly, the filling aid, and the handle for the piston rod safely, in such a way that no third party can injure themselves.

Dispose of used products according to local regulations. For information about correct disposal, contact your local council or authority.

Distributed in the United Kingdom by:

Roche Diabetes Care Limited
Charles Avenue, Basing Hill
Oxfordshire, RG2 6SH, United Kingdom
Accu-Chek Pump Cartridge: 0800 731 22 91
ROI Freephone number: 1 800 882 51
Mobile number: +44 7772 596
www.accu-chek.at

Austria

Roche Diabetes Care Austria Pty. Limited
Insulin Pump Support: 1800 633 457
australia.insulinpump@roche.com
www.accu-chek.com.au

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